

This Kit Includes:



2 Specimen collection swabs



1 collection tube with liquid



1 Biohazard bag with absorbent pad

READ FIRST

CAUTION:


DO NOT touch the white tip of the swab or lay the swab down. If the white tip is touched or the swab is laid down or dropped, your results may not be accurate. If swab gets damaged, use the spare swab included.

DO NOT pre-wet the collection swab with the liquid in the collection tube before collecting a sample.

DO NOT ingest or expose skin/eyes to the liquid in the collection tube. **IF** exposed or concerned, get medical advice/attention.


TO PREVENT REJECTED SPECIMENS, PLEASE READ ALL INSTRUCTIONS BEFORE BEGINNING COLLECTION

1.

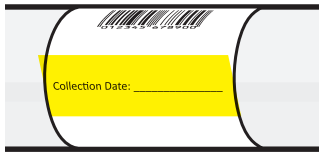


Write the Collection Date on your collection tube. Use MM/DD/YYYY format for the dates (i.e., 08/29/2023).

2.

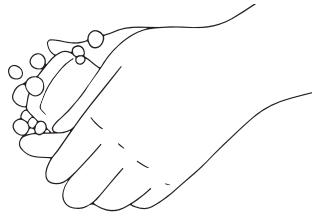


Wash your hands with soap and water thoroughly before proceeding and after completing all steps.

3.

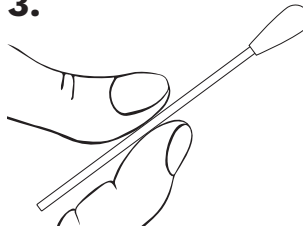


Remove the swab from the wrapper with your clean hands. Hold the swab with the white tip up. Do not touch the tip of the swab to anything.

4.

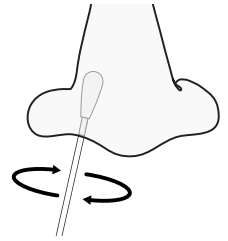


Holding the swab with one hand, gently insert the white tip of the swab about $\frac{1}{2}$ to $\frac{3}{4}$ inches (1 to 2 cm) into the opening of your nose. Rotate the swab for 15 to 30 seconds. Make sure the swab touches the insides of your nose. Remove the swab from your nose being careful not to touch your skin. Do not set the swab down. Repeat in the other nostril.

5.

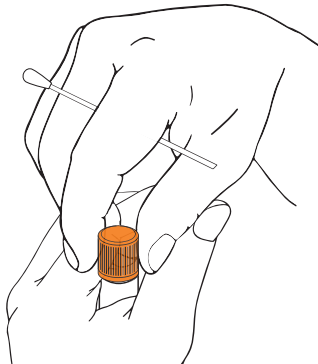


While still holding the swab, unscrew and remove the cap from the transport tube without setting the cap down. Place the swab into the tube with the white tip down, taking great care not to spill or splash the liquid in the collection tube.

6.

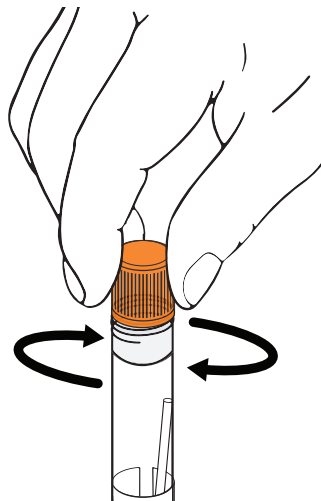

Place lid onto the collection tube and ensure it is closed evenly and tightly.

7.

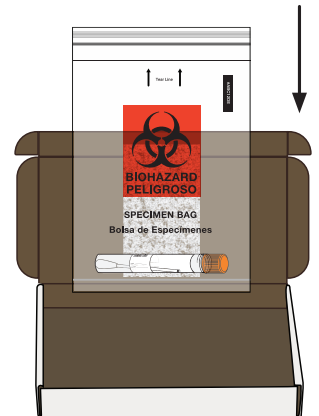

Place specimen into the empty biohazard bag with the absorbent pad and ensure the seal is closed completely. Place biohazard bag in the kit box. Return kit box to kiosk drop box